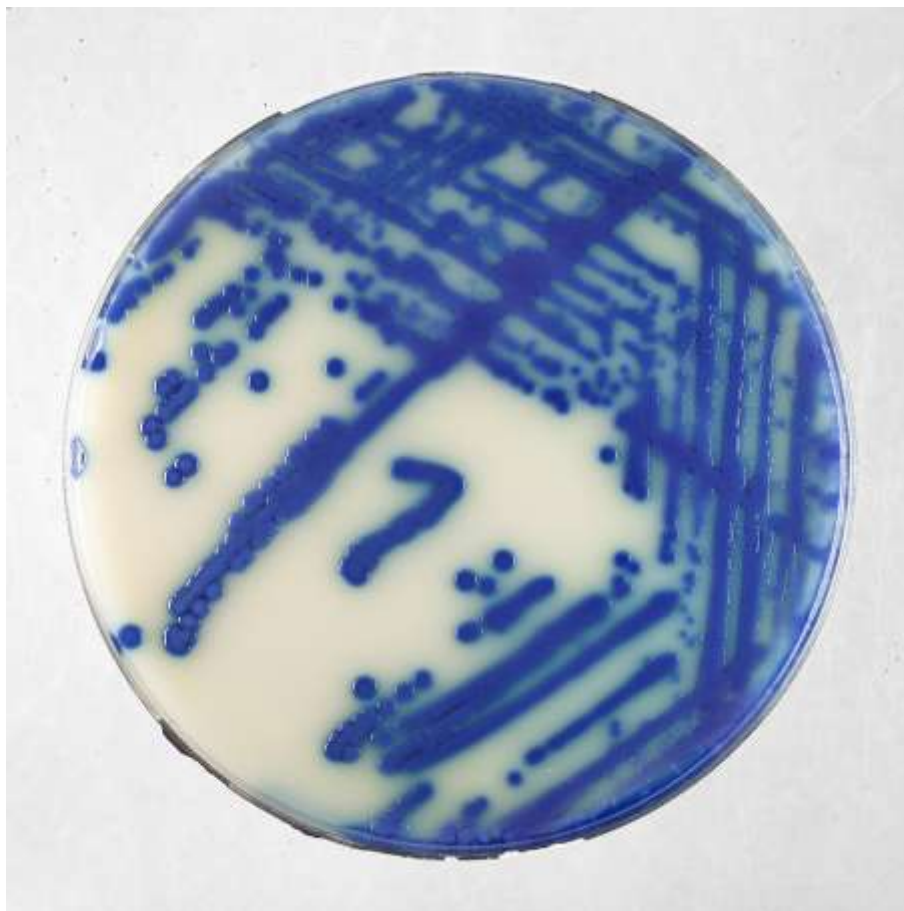
**ChromArt**

**CRE**

**Carbapenem Resistant Enterobacteriaceae**

**Improved chromogenic medium  
for detection of  
Carbapenem-Resistant  
Gram-negative bacteria**

**Biolife**

# ChromArt CRE

**Improved selective differential medium for the detection of Carbapenem Resistant Gram-negative bacteria in clinical specimens**

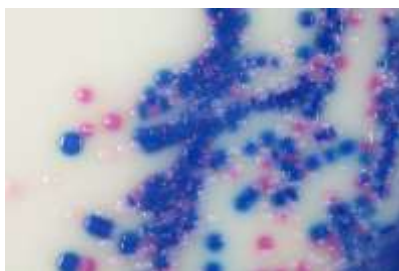
## MAIN CHARACTERISTICS AND ADVANTAGES

- Very good productivity obtained with selected and standardized peptones.
- Enhanced visual differentiation of the colonies due to strong chromatic reactions and opaque contrasting background.
- Specific enzymatic reactions for presumptive identification of main CRE isolates.
- Improved formulation with a very high sensitivity and specificity in the detection of CRE strains (1)


## CRE DETECTION AND DIFFERENTIATION

### SAMPLE PROCESSING

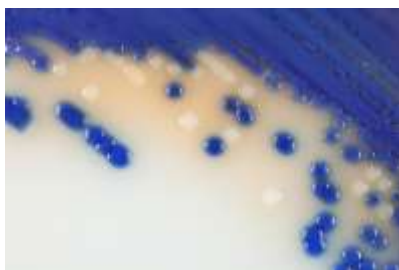- Allow ChromArt CRE plates to warm to room temperature.
- Inoculate ChromArt CRE directly with the faecal specimen or from an isolated colony prepared as a liquid suspension.
- Incubate at 37°C for 18-24 hours in aerobic conditions.
- Reading and interpretation according to the pictures.
- Confirm carbapenemase production using an appropriate method.

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