**ChromArt**

**ESBL**

**Extended Spectrum Beta Lactamase**

**Improved chromogenic medium  
for the detection of  
Extended Spectrum  $\beta$ -Lactamase  
producing Enterobacteria.**

**Biolife**

# ChromArt ESBL

**Improved selective and differential medium for the detection of Extended Spectrum  $\beta$ -Lactamase producing *Enterobacteriaceae*.**


## MAIN CHARACTERISTICS AND ADVANTAGES

- Very good productivity obtained with selected and standardized peptones.
- Enhanced visual differentiation of the colonies due to strong chromatic reactions and to opaque contrasting background.
- Specific enzymatic reactions for presumptive identification of main ESBL isolates.
- Improved formulation with a very high sensitivity and specificity in the detection of ESBL producing Enterobacteria (1).

## ESBL DETECTION AND DIFFERENTIATION

### SAMPLE PROCESSING

- Allow ChromArt ESBL plates to warm to room temperature.
- Inoculate ChromArt ESBL directly with the faecal specimen or urine or other clinical specimens or from an isolated colony prepared as a liquid suspension.
- Incubate at 37°C for 18-24 hours in aerobic conditions.
- Reading and interpretation according to the pictures.
- Confirm ESBL production using an appropriate method.

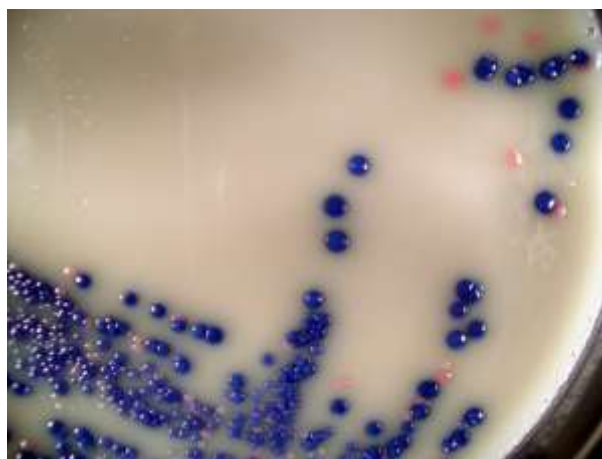**ESBL producing *E.coli* :**  
Pink colonies

**ESBL producing KESC:  
(*Klebsiella, Enterobacter, Serratia, Citrobacter*):**  
Blue/blue-violet colonies

**ESBL producing *Proteus, Morganella, Providencia*:**  
Colonies with brown halo

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### Packaging

REF 548020	ChromArt ESBL	20 ready to use plates (90 mm)
REF 4080252	ChromArt CRE- ESBL Base	500 g (dehydrated medium)
REF 4240080	ChromArt ESBL Supplement	10 x 500 mL